THUASNE

Ortel C4 Rigid

fr	Collier cervical rigide
en	Rigid cervical collar
de	Feste Halskrause
nl	Stijve nekbrace
it	Collare cervicale rigido
es	Collarín cervical rígido
pt	Colar cervical rígido
da	Stiv nakkekrave
fi	Jäykkä kaulatuki
sv	Rigid nackkrage
el	Αυθενικό κολάρο σκληρό
cs	Pevný krční límec
pl	Kołnierz sztywny do unieruchomienia szyi
lv	Stingra kakla ortoze
lt	Standus kaklo įtvaras
et	Jäik emakakaela krae
sl	Opornica za vrat, toga
sk	Anatomicky tvarovaná penová opora
hu	Kemény nyakmerevítő
bg	Твърда цервикална яка
ro	Guler cervical rigid
ru	Жесткий шейный воротник
hr	Tvrda ortoza za vratnu kralježnicu
zh	硬质颈圈
ar	طوق العنق صلب

S	7,50 - 8,75 cm
M	8,75 - 10 cm
L	10 - 11,25 cm
XL	11,25 - 12,50 cm

© Thuasne - 6166102 (2022-03)
© Studio Caterin

THUASNE
120, rue Marius AUFAN
92300 Levallois-Perret
France

www.thuasne.com

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RIGID CERVICAL COLLAR

Description/Intended users

2 anatomically-shaped moulded shells, made of hypoallergenic foam:

-1 anterior shell, with chin sternal support ①

-1 posterior shell, with occipital and upper dorsal support ②

Anterior tracheal opening. ③

Precise self-fastening adjustment. ④

X-ray compatible. ⑤

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Cotton - polyamide - polyethylene - polypropylene.

Properties/Mode of action

Immobilisation of the cervical spine.

Indications

Cervical radiculopathy.

Cervical spondylosis.

Stable cervical fractures.

Post-operative use.

Whiplash.

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of traumatic injuries: severe sprain, unstable fracture.

Do not use for prolonged wear without medical supervision.

Do not use the product if the diagnosis has not been confirmed.

Precautions

Strictly comply with your healthcare professional’s prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, abnormal sensations, consult a healthcare professional.

Choose the appropriate size to fit the patient, referring to the size chart.

Verify the product’s integrity before every use.

It is recommended that a healthcare professional supervises the first application.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device if it is damaged.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support without excessive compression.

Do not wear the product when driving a vehicle.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters,etc.) or wounds of various degrees of severity.

Wearing a cervical collar may cause some weakness of the neck muscles.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Apply the back part of the collar over the patient’s neck. ④

Apply the front part of the collar with the chin resting on the rest provided for this purpose. Center the collar to ensure alignment in a neutral position. ⑥⑥

Close the self-fastener: the front part overlaps the back part. ⑥

Ensure the strap is not fastened too tight. ⑥

Care

See product label. Close the self-fastening tabs before washing. Hand wash. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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