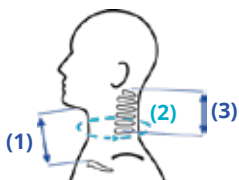
THUASNE®

Cervi-med

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Cervical stabilisation orthosis with a recess in the area of the larynx

Purpose

Cervi-med is a cervical orthosis that surrounds the neck but is open at the larynx. It is used to stabilise the cervical spine. The orthosis may only be used for cervical spine treatment.

Mode of action / Description

Cervi-med is a stabilising orthosis for stabilisation of the cervical spine and to relieve the strain on the musculature. Shaping in the regions of the chin, shoulder and ears, as well as a recess in the area of the larynx ensure optimum fit and high levels of wearing comfort. The excellent wearing comfort is added to by a skin-friendly cotton covering.

Indications

Whiplash; Cervical arthrosis; Sprain; Postoperatively; Torticollis; Cervical syndrome

Contraindications

Skin lesions or diseases in the affected area.

Side effects

None reported when applied and worn as intended.

Putting on the orthosis

1. Open the velcro fastening.
2. Place the orthosis around the neck so that the arrows **(1)** and the „T“ **(2)** on the fastening are pointing upwards.
3. Close the velcro fastening at the back of the neck. The stabilising orthosis should lie comfortably around the neck and be neither too tight nor too loose.

Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Care

Close the velcro fastening before washing. After washing, carefully press out the water and allow to air-dry.

**Material composition**

59 % polyurethane, 28 % cotton, 7 % polyethylene, 6 % polyamide

Please keep this instruction leaflet

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Indications

Coup de fouet cervical; Arthrose cervicale; Entorse; Postopératoire; Torticolis; Syndrome cervical

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- (IL)** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- (S)** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
Thuasne Deutschland GmbH