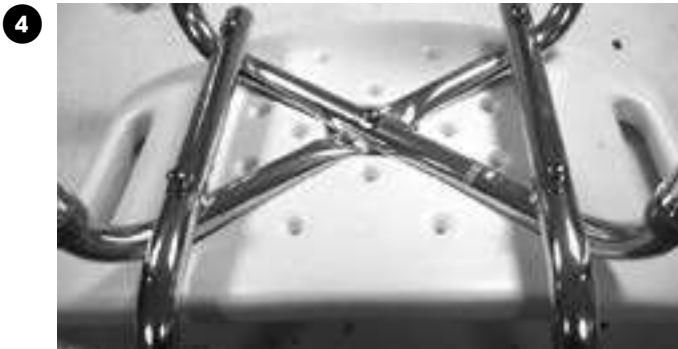
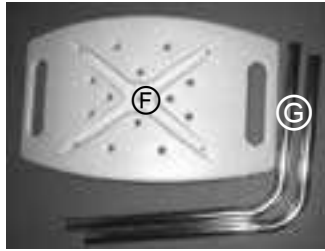
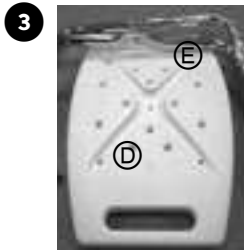
THUASNE

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W1610 - W1600

fr	Tabouret et chaise de douche	4
en	Shower stool and chair	5
de	Duschhocker bzw. -stuhl	6
nl	Douchekruk en -stoel	7
it	Sgabello e sedia da doccia	8
es	Taburete y silla de ducha	10
cs	Stolička a židle do sprchy	11
pl	Taboret i krzesło pod prysznic	12
sk	Sprchovacia stolička a stolček	13
hu	Tusoló ülőke és tusoló szék	14

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Description/Destination

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Description/purpose

Shower stool (W1610) / Shower chair (W1600).

The shower stool and chair provide support when washing, especially in the shower.

Main components

Tubes: aluminum

Ferrules: stainless steel + synthetic rubber

Screws, washers: stainless steel

Push-buttons: brass

Seat, backrest: polyethylene

Indications

Post-operative and post-traumatic support.

Muscle weakness, neuromuscular disorders, stability problems, impaired balance, arthritis of the lower limbs. Degenerative conditions of the lower limbs.

Contraindications

Not suitable for patients weighing > 100 kg.

Do not use in the event of known allergy to any of the components.

Do not use in the event of severe perceptual function deficit or severe balance disorders.

Precautions

Important warnings

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Clean the product thoroughly before first use.

Before using this product, check that:

- it is stable and rigid,
- all components are correctly attached and properly screwed in place.

The healthcare professional who supplied the product must make sure that it is appropriate for the patient.

It is recommended that a healthcare professional assembles and adjusts the device and supervises the first use.

Follow the advice of the healthcare professional who prescribed or supplied the product and consult them in case of doubt.

Do not use the product for any purpose other than its intended use: risk of injury or damage to equipment.

All four feet must always be in contact with the floor and the floor must be level.

Take care to sit in the middle of the seat.

Do not pick up or move the product while the user is sitting on it: risk of injury or damage to equipment.

Be aware of slippery surfaces.

Monitor the use of the product by children, people with limited physical capacity or people with cognitive impairments.

Never allow children to play with the product.

Do not make any mechanical changes to the product, as this could compromise its safety. Only original spare parts may be used.

Undesirable side-effects

Risk of discomfort in the event of incorrect adjustment of the product.

Danger of falling or loss of balance, which can lead to serious injuries.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use

Assembling the stool:

The shower stool consists of the following components: 1 seat **A**, 2 legs **B** and 5 screws with washers (40 mm long) **C**. **1**

Check that all the parts are present.

Place the seat upside down on a table and attach the legs with the five screws supplied, using a cross-head screwdriver. **2**

Assembling the chair:

The shower chair consists of the following components: 1 seat **D**, 2 legs **E**, 1 backrest **F**, 2 backrest supports **G** and 9 screws of different lengths with washers: 4 of 62 mm **H**, 4 of 46 mm **I** and 1 of 40 mm **J**. **3**

Check that all the parts are present.

Use the 62 mm screws to attach the legs to the backrest tubes using a cross-head screwdriver. The 40 mm screw is positioned in the middle. **4**

The 46 mm screws attach the backrest to the tubes. **5**

Adjusting the height

Press the push-button while pulling the tube in order to adjust the height.

Check that the push-button is properly engaged in the hole corresponding to the desired setting.

Make sure that all four feet are adjusted to the same height.

Maintenance and care

Maintenance

If any part of the product becomes detached, loose or visibly damaged, contact a specialized dealer or healthcare professional.

If necessary, replacement parts and original accessories are available from your Thuasne product dealer.

Care

Clean regularly with a mild, non-abrasive detergent.

Dry with a soft cloth.

In an institution, follow the recommendations of the hygiene manager.

Storage

Do not expose the product to extreme temperatures.

Disposal

Dispose of in accordance with local regulations.

Technical characteristics

Stool

<p>Seat dimensions</p>  <p>51 x 31 cm</p>	<p>Adjustable seat height</p>  <p>34 - 46 cm</p>	<p>Weight</p>  <p>2 kg</p>
<p>Maximum user weight</p>  <p>100 kg max</p>		

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©Thuasne - 2033601 (2021-12)