

THUASNE®

en	Sport Elbow support	4
fr	Coudière de maintien sport	5
de	Sport-Ellenbogenbandage	6
nl	Steunbrace voor de elleboog sport	7
it	Gomitiera di sostegno per uso sportivo	8
es	Codera de sujeción para uso deportivo	9
pt	Cotoveleira de suporte para uso desportivo	10
da	Albuestøttebind sport	12
fi	Kuynärpäätukki urheilukäyttöön	13
sv	Armbågsstöd sport	14
el	Αθλητική περιαιγκωνίδα συγκράτησης	15
cs	Loketní bandáž sport	16
pl	Opaska podtrzymująca łokieć sportowa	17
lv	Sporta elkoņa atbalsts	18
lt	Palaikomasis alkūnės įtvaras sportui	19
et	Spordi küünarnukiortoos	20
sl	Opornica za komolec šport	21
sk	Športová bandáž na lakeť	22
hu	Könyökrögzítő sportoláshoz	23
bg	Поддържащ налакътник за спорт	24
ro	Cotieră de susținere sport	26
hr	Sportski stezник za potporu lakta	27

SPORT

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Istrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu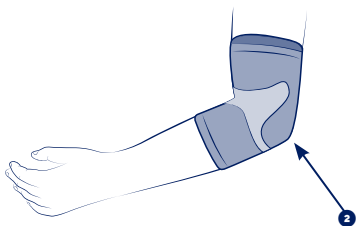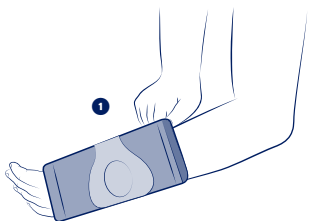

 <p>8 cm</p>
<p>cm</p>
<p>S</p> <p>22 - 24</p>
<p>M</p> <p>25 - 27</p>
<p>L</p> <p>28 - 30</p>
<p>XL</p> <p>31 - 33</p>

en	Forearm circumference
fr	Tour de l'avant-bras
de	Umfang des Unterarms
nl	Omtrek van de voorarm
it	Circonferenza dell'avambraccio
es	Circunferencia del antebrazo
pt	Perímetro do antebraço
da	Omkreds af underarm
fi	Yläkäsivarren ympärysmitta
sv	Runt underarmen
el	Περίμετρος του αντιβραχίου
cs	Pro předloktí
pl	Opaska na przedramię
lv	Apkārt apakšdelmam
lt	Dilbio apimtis
et	Küünarvarre ümbermõõt
sl	Obseg podlakti
sk	Obvod predlaktia
hu	Alkar kerülete
bg	Обиколка на предмишницата
ro	Circumferința antebrațului
hr	Opseg podlaktice

en

SPORT ELBOW SUPPORT

Support level 1

Description/Destination

This device is an elbow brace providing joint support and stabilization.

This device can be used when resuming sports activities.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Textile components:

Polyamide - elastane - cotton.

Properties/Mode of action

Even compression on the joint.

Improved proprioception.

Indications

Treatment of lateral or medial epicondylitis (tennis/golfer's elbow).

Prevention of elbow tendinopathy recurrence.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Pull the product on (the logo indicates the top and front of the product). ❶

Make sure the elbow is properly centered in the zone provided for this purpose. ❷

Garment care

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2051901 (2024-02)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186