en

## **SPORT THIGH SUPPORT**

*Support level 1*

### **Description/Intended use**

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. Bilateral product.

Available in 4 sizes.

The device is composed of: a synthetic rubber (SBR) tissue.

This device can be used when resuming sports activities.

### **Composition**

Synthetic rubber (SBR) - polyamide - elastane.

### **Properties/Mode of action**

Fabric allowing an even compression.

Muscle support.

Limitation of the heat loss by the synthetic rubber (SBR).

### **Indications**

Muscular thigh injuries.

### **Contraindications**

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

### **Precautions**

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional in case of uncertain diagnosis.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

In case of doubt before any sports activity, consult a healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

### **Undesirable side-effect**

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### **Instructions for use/Application**

It is recommended to wear the device directly on the skin, unless contraindicated.

Put the product on like a sock. Position it on the thigh, the Thuasne logo must be on the upper part, centered and centered in relation to the patella.

#### **Garment care**

Hand washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

#### **Storage**

Store at room temperature, preferably in the original packaging.

#### **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2053101 (2024-03)
**THUASNE UK Ltd**
Thuasne SAS – SIREN/RCS Nanterre 542 091 186