THUASNE

Immo

fr	Écharpe d'immobilisation d'épaule	5
en	Shoulder immobilisation sling	6
de	Tuch zur Ruhigstellung der Schulter	7
nl	Sling voor de immobilisatie van de schouder	8
it	Fascia di immobilizzazione della spalla	10
es	Cabestrillo para inmovilización del hombro	11
pt	Suportede imobilização do ombro	12
da	Halstørklæde til skulderimmobilisering	13
fi	Olkapään immobilisaatiokantoside	14
sv	Halsduk för att immobilisera axeln	16
el	Φάκελος ακινητοποίησης ώμου	17
cs	Imobilizační ramenní závěs	18
pl	Szal do unieruchomienia barku	19
lv	Pleca imobilizācijas soma	20
lt	Peties imobilizavimo įtvaras	21
et	Õlaliigest fikseeriv side	22
sl	Zanka za imobilizacijo rame	24
sk	Ramenný imobilizačný záves	25
hu	Vállrögzítő heveder	26
bg	Бандаж за обездвижване на рамото	27
ro	Orteză „tip eşarfă” de imobilizare a umărului	28
ru	Повязка для обездвиживания плеча	29
zh	肩部固定圍巾	31

	50 - 71 cm	1
	72 - 94 cm	2
	95 - 117 cm	3
	118 - 140 cm	4

fr	Tour de taille
en	Waist measurement
de	Tailenumfang
nl	Tailleomvang
it	Circonferenza vita
es	Contorno de cintura
pt	Perímetro de cintura
da	Taljemål
fi	Vyötärönympärys
sv	Midjemått
el	Περίμετρος μέσης
cs	Obvod pasu
pl	Obwód w talii
lv	Vidukļa apkārtmērs
lt	Juosmens apimtis
et	Vööübermõõt
sl	Obseg pasu
sk	Obvod pása
hu	Derékbőség
bg	Обиколка на талията
ro	Circumferința taliei
ru	Обхват талии
zh	腰围

Immobilisation
Immobilisation
Ruhigstellung
Immobilisatie
Immobilizzazione
Inmovilización
Imobilização
Støtte
Immobilisoi
Immobilisering
Ακινητοποίηση
Znehybnění
Unieruchomienie
Imobilizācija
Imobilizavimas
Liikumatuks muutmine
Imobilizacija
Imobilizácia
Rögzítés
Обездвижане
Imobilizare
Иммобилизация
固定

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SHOULDER IMMOBILISATION SLING

Description/Use

This device is a shoulder sling allowing the immobilisation of the shoulder and/or the elbow in neutral position.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

viscose - polyester - polyurethane - steel - polyamide - polyoxymethylene - polypropylene - artificial rubber

Properties/Mode of action

Adjustable structure allowing immobilisation in neutral position:

Reinforced abdominal belt. ①

Support armbands for the forearm. ②

Straps for tightening the armbands. ③

Fastening strap for the fingers. ④

Removable fastening and elbow support bandage. ⑤

Shoulder brace ⑥ ⑦ with thoracic fastening strap ⑧ ⑨.

Bilateral product.

Indications

Shoulder and/or elbow injuries or inflammatory conditions.

Conservative treatment for non-surgical proximal humeral fractures.

Shoulder dislocation.

Non surgical fractures of the clavícula and/or scapula. Post-operative immobilization (such as after rotator cuff surgery or dislocation reduction...).

Shoulder subluxation or dislocation prevention (joint instability).

Support for paralytic upper limb.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of complex or surgical shoulder fractures or rib cage fractures.

Do not use in certain postoperative conditions (multiple fractures, pin insertion, open fractures, etc.).

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not bring the product into direct contact with greasy substances (ointment, cream, etc.).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Do not wear the product when driving a vehicle.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Belt: diagram I

- Using your healthy arm, place the belt ① around your waist, after having positioned your other arm in the armbands ② without fastening them, and fasten. You can now fasten the armbands using the straps. For the right arm, close the upper part over the lower part. ③ ④

For the left arm, close the lower part over the upper part. ③ ⑥

2. Check that the belt is tight enough for optimum arm support and immobilisation. If the arm does not feel sufficiently supported, re-adjust the straps.

Removable fastening bandages: diagram II

3. The small removable strap ④ is used to make sure your hand is comfortably supported. Check that it circles the belt and slide it to the ends of your fingers.
4. The large removable bandage ⑤ : check that one of the self-fastening tabs of this removable fastening bandage for the elbow is already attached to the belt, about twenty centimetres behind the large armband ②. The professional supplying the product will be able to help you position the bandage properly. Get hold of it with your healthy hand and take it forward and clove over the nearest armband to cover, support and immobilise the elbow.

Shoulder brace: diagram III

5. Position the shoulder brace ③ ④ ⑤ on the shoulder to be immobilised, with the thoracic fastening strap ⑥ ⑦ already attached to the back buckle on the shoulder brace, wrap the self-fastening bandage around the chest and through the front buckle of the shoulder brace and attach it to itself.

Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Close the self-fastening tabs before washing. Machine washable at 40 °C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2027701 (2021-04)