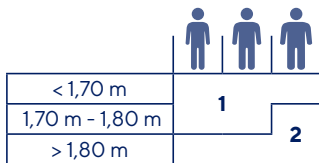
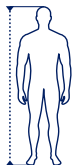
THUASNE

ImmoVest

fr	Gilet d'immobilisation d'épaule	5
en	Shoulder immobilisation vest	6
de	Bandage zur Ruhigstellung der Schulter	7
nl	Gordel voor de immobilisatie van de schouder	9
it	Gilè di immobilizzazione della spalla	10
es	Chaleco para inmovilización del hombro	11
pt	Colete de imobilização de ombro	13
da	Vest skulderimmobilisering	14
fi	Olkapään immobilisaatiolivi	15
sv	Väst för att immobilisera axeln	16
el	Γιλέκο ακινητοποίησης ώμου	17
cs	Vesta pro imobilizaci ramene	19
pl	Orteza do unieruchomienia barku	20
lv	Pleca imobilizācijas veste	21
lt	Peties imobilizavimo liemenė	22
et	Õlaliigest fikseeriv ortoos	24
sl	Steznik za imobilizacija rame	25
sk	Vesta na znehybnenie ramena	26
hu	Vállrögzítő mellény	27
bg	Жилетка за обездвижване на рамото	28
ro	Orteză „tip vestă” de imobilizare a umărului	30
ru	Жилет для обездвиживания плеча	31
zh	肩部固定背心	33

fr	Taille du patient/corpulence
en	Patient height/corpulence
de	Größe des Patienten/Leibesumfang
nl	Maat van de patiënt/lichaamsbouw
it	Altezza del paziente/corpulenza
es	Tamaño del paciente/corpulencia
pt	Tamanho do paciente/corpulência
da	Patientstørrelse/opbygning
fi	Potilaan koko/ruumiinrakenne
sv	Patientens storlek/korpulens
el	Ύψος ασθενούς/σωματότυπος
cs	Výška pacienta/stavba těla
pl	Rozmiar pacjenta/budowa ciała
lv	Pacienta augums / svars
lt	Paciento dydis/kūno sudėjimas
et	Patsiendi mõõdud/kehaehitus
sl	Velikost bolnika/postava
sk	Výška pacienta/veľkosť postavy
hu	Beteg mérete/testalkata
bg	Размер на пациента/телосложение
ro	Talia pacientului/corpulență
ru	Размер пациента/телосложение
zh	患者身高/体重

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Immobilisatie

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Støtte

Immobilisoi

Immobilisering

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SHOULDER IMMOBILISATION VEST

Description/Use

This device is a shoulder sling allowing the immobilisation of the shoulder and/or the elbow in neutral position.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

polyamide - polyurethane - polyester - elastane

Properties/Mode of action

Adjustable structure allowing immobilisation in neutral position.

Anatomical and wraparound structure for reinforced protection.

Bilateral product.

Indications

Shoulder and/or elbow injuries or inflammatory conditions.

Conservative treatment for non-surgical proximal humeral fractures.

Shoulder dislocation.

Non surgical fractures of the clavícula and/or scapula. Post-operative immobilization (such as after rotator cuff surgery or dislocation reduction...).

Shoulder subluxation or dislocation prevention (joint instability).

Support for paralytic upper limb.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of complex or surgical shoulder fractures or rib cage fractures.

Do not use in certain postoperative conditions (multiple fractures, pin insertion, open fractures, etc.).

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not bring the product into direct contact with greasy substances (ointment, cream, etc.).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Do not wear the product when driving a vehicle.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

To make fitting easier, carefully follow the numbers marked on the straps and the grey (left side) and black (right side) colours.

It should be noted that the help of another person is needed when first fitting the product.

For the left shoulder:

STEP 1 : Fold the ends of the 3 straps back on themselves, covering the black self-fastening areas.

STEP 2 : Place the fleecy pad on the shoulder, with the grey seam visible on the outside.

STEP 3 : Position the arm at a right angle over the vest. Support it by raising the **Left/Gauche 1** strap and fasten this strap at the shoulder. Then raise the **Left/Gauche 2** strap diagonally, to firmly support the hand.

N.B.: make sure the fleecy pad is properly centred on the shoulder.

STEP 4 : Take hold of the **Left/Gauche 3** strap, fasten it on the forearm, then take it towards the back, attaching it a first time using the pivoting hook of the shoulder brace, then fastening it onto the shoulder brace with the hook of the strap.

STEP 5 : Take hold of the **Left/Gauche 4** strap, then take it to the front of the body and fasten it on the forearm, holding the elbow.

STEP 6 : Adjust the shoulder.

Do not hesitate to readjust the product several times to ensure perfect immobilisation.

For the right shoulder:

STEP 1 : Fold the ends of the 3 straps back on themselves, covering the grey self-fastening areas.

STEP 2 : Place the fleecy pad on the shoulder, with the black seam visible on the outside.

STEP 3 : Position the arm at a right angle over the vest. Support it by raising the **Right/Droit 1** strap and fasten this strap at the shoulder. Then raise the **Right/Droit 2** strap diagonally, to firmly support the hand.

N.B.: make sure the fleecy pad is properly centred on the shoulder.

STEP 4 : Take hold of the **Right/Droit 3** strap, fasten it on the forearm, then take it towards the back, attaching it a first time using the pivoting hook of the shoulder brace, then fastening it onto the shoulder brace with the hook of the strap.

STEP 5 : Take hold of the **Right/Droit 4** strap, then take it to the front of the body and fasten it on the forearm, holding the elbow.

STEP 6 : Adjust the shoulder.

Do not hesitate to readjust the product several times to ensure perfect immobilisation.

Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Close the self-fastening tabs before washing. Machine washable at 40 °C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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Polyamid - Polyurethan - Polyester - Elasthan

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©Thuasne - 2027801 (2021-04)