THUASNE

Ligastrap® Immo

fr	Attelle stabilisatrice de cheville avec sangles de renfort	5
en	Stabilising ankle brace with support strapping	6
de	Orthese zur Stabilisierung des Sprunggelenks mit Stützgurten	8
nl	Stabilisatiebrace voor de enkel met steunbanden	10
it	Tutore stabilizzatore per caviglia con fasce di rinforzo	12
es	Férula estabilizadora de tobillo con cintas de refuerzo	14
pt	Tala estabilizadora do tornozelo com fitas de reforço	16
da	Stabiliserende ankelskinne med støttende stropper	17
fi	Stabiloiva nilkkatuki vahvistushihnoilla	19
sv	Stabiliseringsskena för ankeln med förstärkningsremmar	21
el	Νάρθηκας σταθεροποίησης ποδοκνημικής με ιμάντες ενίσχυσης	23
cs	Stabilizační ortéza kotníku s fixačními popruhy	24
pl	Orteza do stabilizacji kostki z paskami wzmacniającymi	26
lv	Potītes stabilizēšanas saite ar nostiprināšanas siksnām	28
lt	Stabilizuojantis kulkšnies įtvartas su sutvirtinimo diržais	30
et	Hüppeliigese fikseerimise lahas tugirihmadega	31
sl	Opornica za stabilizacijo gležnja z ojačitvenimi trakovi	33
sk	Stabilizačná členková dlaha so spevňujúcimi pásmi	35
hu	Stabilizáló rögzítő bokához erősítő szíjakkal	36
bg	Стабилизираща шина за глезен с каишки за подсилване	38
ro	Atelă stabilizatoare pentru gleznă, cu chingi de ranforsare	40
ru	Стабилизирующий ортез для лодыжки с усиливающими ремнями	42
hr	Udlaga za stabilizaciju gležnja s remenima za ojačanje	44
zh	帶加固帶的腳踝穩定夾板	46
ar	جبيرة تثبيت الكاحل مع أحزمة تقوية	47

	< 160 cm	0
	> 160 cm	1

fr	Taille du patient	Stabilisation
en	Patient height	Stabilization
de	Größe des Patienten	Stabilisierung
nl	Maat van de patiënt	Stabilisatie
it	Altezza del paziente	Stabilizzazione
es	Tamaño del paciente	Estabilización
pt	Tamanho do paciente	Estabilização
da	Patientens højde	Stabilisering
fi	Potilaan koko	Stabiloi
sv	Patientens storlek	Stabilisering
el	Μέγεθος ασθενούς	Σταθεροποίηση
cs	Výška pacienta	Stabilizace
pl	Rozmiar pacjenta	Stabilizacja
lv	Pacienta augums	Stabilizēšana
lt	Paciento dydis	Stabilizavimas
et	Patsiendi suurus	Stabiliseerimine
sl	Velikost pacienta	Stabiliziranje
sk	Veľkosť pacienta	Stabilizácia
hu	A beteg mérete	Stabilizálás
bg	Размер на пациента	Стабилизация
ro	Talia pacientului	Stabilizare
ru	Размер пациента	Стабилизация
hr	Veličina pacijenta	Stabilizacija
zh	患者身高	稳定
ar	مقاس المريض	الاستقرار

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Conserver cette notice.

en

STABILISING ANKLE BRACE WITH SUPPORT STRAPPING

Description/Destination

This device is a ankle brace providing joint stabilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Rigid components: Polypropylene - Polyester - Polyamide - Polyurethane - Ethylene Vinyl Acetate.

Textile components: Cotton - Polyester - Elastodiene.

Properties/Mode of action

Rigid plastic lateral shells.

Precise fit thanks to adjustable straps.

Heel loop with width adjustment.

Removable straps colour coded for fit, improving the reduction of oedema.

Indications

Moderate to severe ankle sprain.

Ankle immobilisation.

Conservative treatment of Achilles tendon injuries.

Post-operative, post-traumatic residual pain and/or edema.

Sport activities resumption.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

History of venous or lymphatic disorders.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a health care professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

Check the condition of the affected limb daily (with particular attention for patients with sensory deficit).

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a health care professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Do not wear the product in a medical imaging device.

Do not wear the product when driving a vehicle.

The systematic use of a long sock is recommended when wearing the device.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Possible risk of ulcers.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

From a sitting position pull on sock with your foot on the floor.

- 1 Adjust the height of the heel strap so that the malleoli are positioned in the cut-outs (use the marker on the strap to properly position the orthosis).

- 2 Place the centre of the strapping on the inside of the brace heel. This strap must be parallel to the product, placed flat. Then place your heel on the orthosis.
- 3 Pull one end of the strap, take it over the top of the foot and around the ankle and fasten it to the self-fastening zone at its base.
Repeat the same operation with the second end of the strap. Attach the two black self-fastening zone together and the blue self-fastening end to the blue-stitched self-fastening zone.
- 4 Pull up the two side shells.
Fasten the upper strap by wrapping it around the leg and through the buckle, and attaching it with the self-fastening tab.
- 5 Repeat the operation with the lower strap, taking care to use the double pulley system properly.

Carefully slip your foot into a shoe.

The ankle brace may be taken off to allow the foot to be washed.

Only put it back on once you have properly dried the foot again.

Care/Maintenance

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Wipe with a damp cloth. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water.

Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2021101 (2020-11)