THUASNE

Argicalm®

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Conserver cette notice.

en

HOT AND COLD PACK - COLD MASK - REUSABLE CLAY PACK

Description/Destination

Reusable clay pack. **1**

Removable protected sleeve and comfortable. Velvet effect. **2**

Self-adhesive elastic fastening strap. **3**

The device is intended only for the treatment of the indications listed in adults, adolescents and children.

Composition

Cotton, elastane, polyester, clay, PVC

Properties/Mode of action

- Use cold : soothes pain, reduces inflammation, bruises and edema/swelling.
- Use hot : soothes pain, increases blood flow, relaxes.

Indications

- Use cold : sprains, strains, tendinitis, migraines/headache, eye pain, toothache/dental pain, bruises, arthritis, maxillofacial surgery and eye surgery.
- Use hot : Back pain, contractures/muscle and articular stiffness, cramps, torticollis (stiff neck), abdominal pain.

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

In case of diabetes or vascular and circulatory problems, consult a doctor before using this product.

Do not use the cold mode in case of hypersensitivity to cold.

Do not use with analgesic and/or anesthetic products (such as creams, balms, lotions, liniments...).

Precautions

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

The use of a medical device by a child, should be done under the supervision of an adult or a healthcare professional.

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Do not apply directly to the skin, use the provided sleeve.

External use only, do not ingest contents.

In case of accidental contact, rinse thoroughly, drink water if swallowed.

Do not heat up the Argicalm® clay pack using a microwave or a bain-marie if it is not already at room temperature.

Do not heat Argicalm® clay packs beyond the duration and associated power indicated in the present instructions for use: the clay pack could be damaged or even destroyed.

If the power of the microwave is unknown, use the lowest power.

Be careful not to burn yourself when handling the Argicalm® clay pack with bare hands in case of hot water or microwave use.

If the Argicalm® clay pack is too hot, wait until it cools down before using it in its sleeve.

Always check clay pack temperature before application, prolonged application beyond provided indications may cause injuries including burns.

Do not place excessive weight, sit, or lie down on the pack.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

- Use cold  : Instructions  :

Remove the clay pack Argicalm® from its transparent plastic bag.

Place it flat in a freezer compartment (-20 °C) for at least 2 hours before use.

For more comfort, massage/knead the Argicalm® clay pack to make it more flexible before inserting it in its sleeve.

If the Argicalm® clay pack is too cold or too solid, leave it for a few minutes at room temperature before using it.

To avoid the risk of cold injuries, do not apply on body for more than 20 min and wait at least 20 min before applying again.

• Use hot.

Instructions for use in the microwave **B** :

Remove the Argicalm® clay pack from its transparent plastic bag.

Place it flat at room temperature in the microwave and follow the heating protocol.

Massage the clay pack to spread warmth evenly before inserting it in its sleeve.

Instructions for use in a bain-marie **C** :

Heat water to boiling point.

Make sure the volume is sufficient to fully immerse the Argicalm® clay pack.

Turn off the heat source.

Remove the clay pack Argicalm® from its transparent plastic bag.

Insert the Argicalm® clay pack at room temperature into the hot water for from 5 to a maximum of 20 minutes (depending on the clay pack weight) until it reaches the required temperature.

Care

Hand washable with soft detergent. Do not dry clean. Do not apply solvent or alcohol to the pack. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Due to the natural origin of the clay contained in Argicalm®, the appearance of the clay pack may change during the life of the product (colored dots or marble-like areas).

This change in color has no impact on the function or performance of the Argicalm® clay packs.

Disposal

Dispose of in accordance with local regulations.

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Baumwolle, Elasthan, Polyester, Lehm, PVC

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©Thuasne - 2028301 (2021-03)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186 – capital 1 950 000 euros